


1	Perform quick assessment of functional mobility a) Observe the user performing the following tasks: i. Walk at their normal pace for 10 steps ii. Sit-to-stand and stand-to-sit 4 times iii. Go up and down a flight of stairs (if appropriate) b) Observe other movements of interest (base on user goals/need) c) Write a summary of user deficits on FO-8007
2	Set clinical goals (FO-8007)  a) Base goals on initial assessment. The goals can be vague or precise (e.g. improve squatting or improve knee flexion at pre-swing of walking gait) b) Write clinical goals on FO-8007
3	Pick an analysis method Select appropriate method to detect deficits <i>related to clinical goals</i>  The use of a recorded form is <i>recommended</i> such as: gait analysis checklist, videos, timed function tests, etc. Write the name of the selected method on FO-8007
4	Collect baseline data using chosen method
5	Review and prioritize baseline data a) Check baseline data to make sure deficit is detected b) <u>Prioritize deficits related to clinical goal(s) for fine-tuning of Keeogo profile</u> c) Record prioritized list of deficits on FO-8007
6	Fit, familiarize, and fine tune Keeogo profile  a) Fit Keeogo using Clinician Fitting Checklist (IFU-001) b) Complete instruction Sheets #1-6 to tune by prioritized deficits

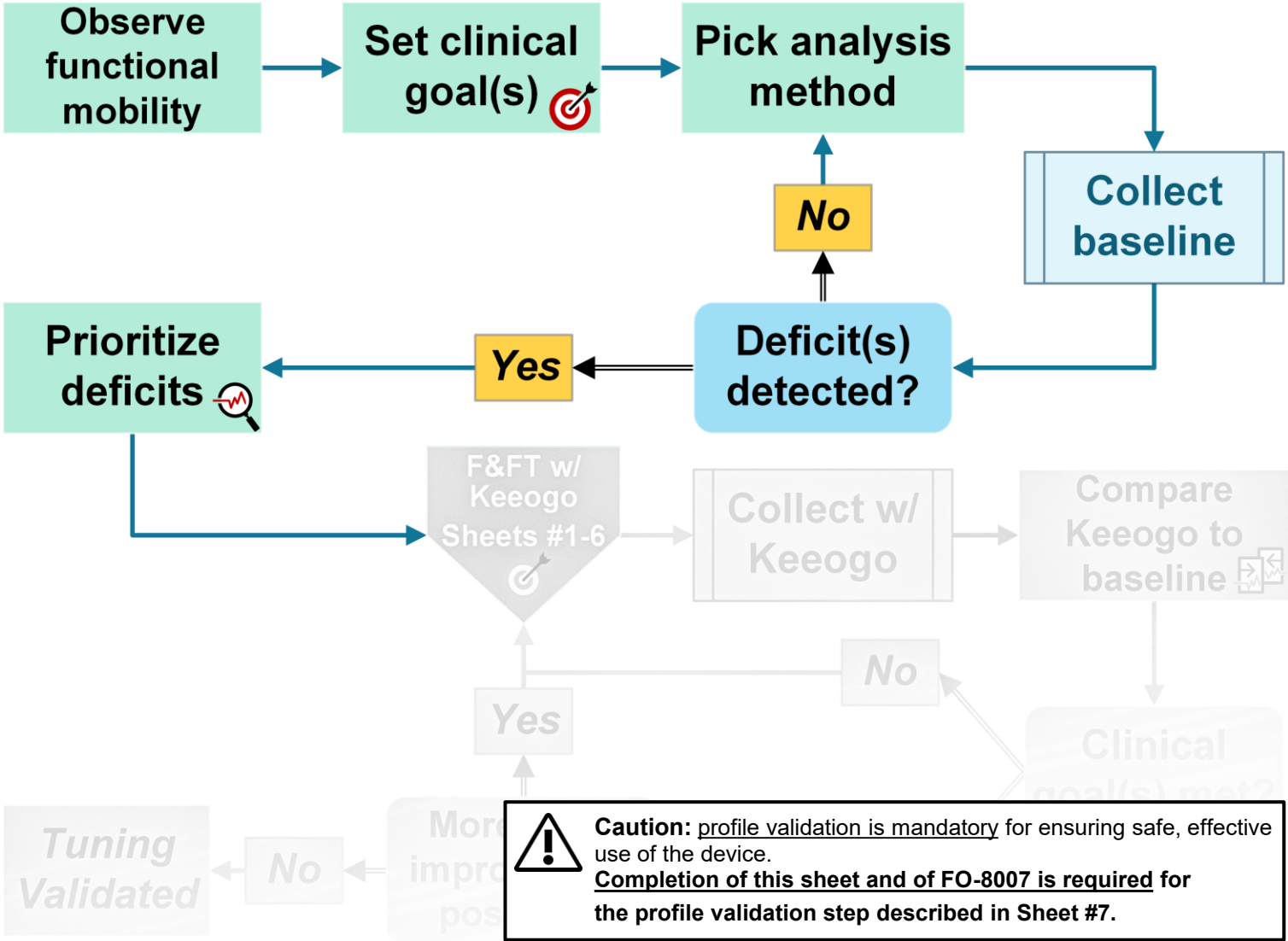

Blue Star Training Module 3 Instruction Sheet # 0	EDU-8013 Blue Star Module 3.0- Identifying Targets of Intervention with Keeogo			Setting:	Rehab and clinical	Doc #:	EDU-8013
				Supervision:	Certified clinical professional	Version:	1,2
	Population:	General use (per indications)	Section:	Clinical Tools	Operation:	Identifying targets of intervention with Keeogo	

Revision History			
DATE	REV	Change description	Owner
2019-04-09	-	Original Version (M. Brookshaw)	B-Temia Inc.
2019-06-17	-	Updated format (M. Brookshaw)	
2019-06-30	-	Revised for simplicity (M. Brookshaw and A. Mai)	
2019-07-15	-	Updated wording for clarity (M. Brookshaw and A. Mai)	
2019-07-29	1,0	Modified for general use training and implementation into QMS	
2019-11-21	1,1	Modified for general use training	
2020-05-11	1,2	Eliminated reference to FO-8004 document (A.Mai)	
This document SHALL be print on both side to show the revision history on the back side			

